


The NavDx[®] Test Result: Surveillance

This easy-to-interpret, actionable test result provides valuable information that informs clinical decisions.

The highly sensitive NavDx test finds and measures *tumor tissue modified viral* HPV DNA (TTMV-HPV DNA) in a patient's blood sample. TTMV-HPV DNA is an HPV-driven cancer specific biomarker, found only in people with active HPV-driven cancer. The test analyzes fragments of TTMV-HPV DNA released by tumor cells into the blood to provide a TTMV Score, reflective of the normalized TTMV-HPV DNA fragments/ml of plasma.¹

The TTMV-HPV DNA biomarker is also specific to the HPV genotype involved in the patient's cancer. The test detects HPV-16, HPV-18, HPV-31, HPV-33, and HPV-35. *The NavDx test is not an HPV test; it does not detect active HPV infection that is unrelated to an HPV+ cancer.*

Once a specimen arrives at our laboratory, NavDx results are typically available within 7 business days. You will receive notification through the Provider Portal when results are ready.

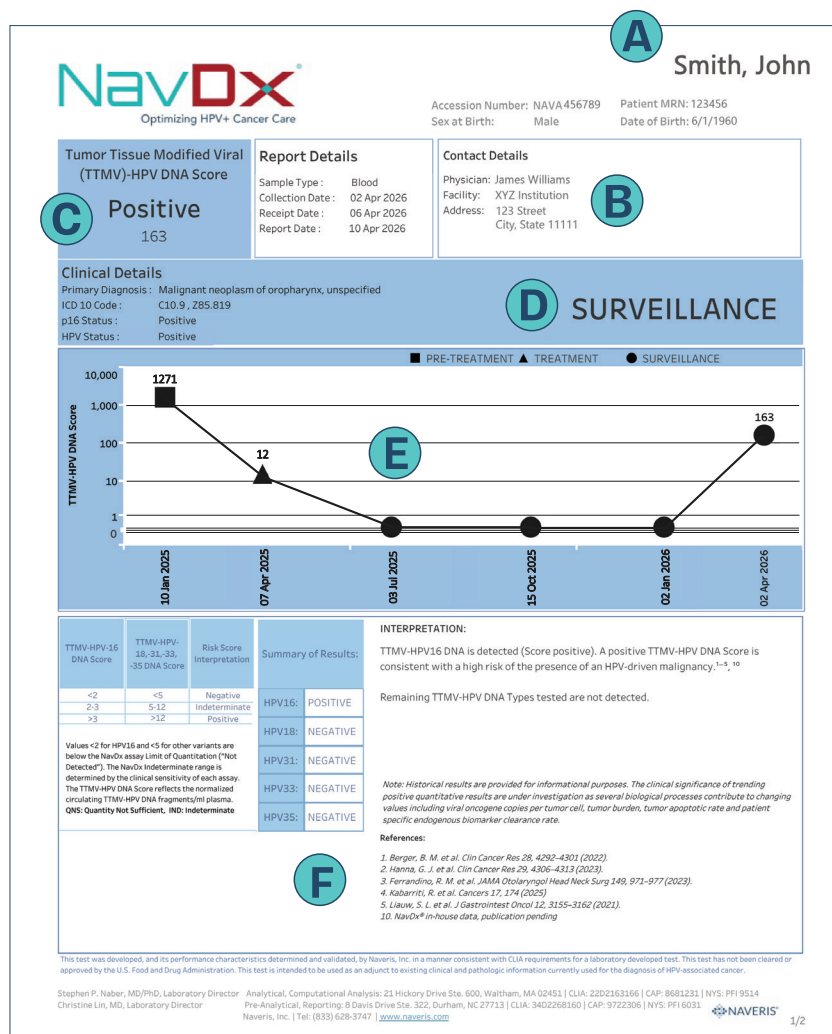

Each NavDx report includes:

- A** Patient information
- B** Provider information
- C** Quick summary of test findings: TTMV-HPV DNA status and TTMV Score
- D** Clinical details: Summarizes the patient's key clinical information at the time the test was ordered
- E** The graph: Sequentially plots the patient's TTMV Score history
- F** Test results & interpretation: Includes TTMV Score, Risk Score Interpretation, tumor HPV genotype, and a brief qualitative, written interpretation of the test result.